ENGLISH

For use with ADAMS A1c

CALIBRATOR 90

Intended Use

Calibration of ADAMS A1c HA-8190V.

Warnings and Precautions

 Handle this product and sample as biohazardous material.

WARNING:

This product is produced with human blood which has tested negative for HBs antigen, HIV-1 antibody, HIV-2 antibody and HCV antibody. However, there is no test that definitively screen for infectivity. Handle the products with the same care as in handling patient samples.

Personal Protection

When handling this product or sample, wear protective gloves to prevent exposure to pathogenic microbes.

First Aid

If this product or sample makes contact with skin, eyes or mouth, immediately wash with plenty of water and take other appropriate first-aid measures. Consult a doctor.

Reconstitution

1. Remove the calibrator and a special diluent from the refrigerator and allow them to adjust to the measurement temperature for approximately 10 minutes.
2. Pipette exactly 3 mL of diluent to each of the calibrator bottles and close the caps.
3. Leave the bottles for approximately 10 minutes in the measurement temperature, and then gently invert them to mix carefully to avoid air bubbles forming until the calibrator is completely dissolved.

NOTE:

- Exercise care to avoid air bubbles forming while pipetting and / or mixing.
- **Do not** use the product if it is cloudy or contaminated with bacteria after dissolution.
- Store the product at a temperature between 2°C and 8°C after dissolution, and use within 8 hours.

Storage

2 - 8°C

Testing Procedure

Operation Procedure

IMPORTANT:

- **Do not** use this product after its expiration date.

For detailed instructions, see the operating manual for ADAMS A1c HA-8190V.

Standard Values

The standard value list includes the following 4 kinds of standard value. Be sure to check the appropriate standard value before use.

IFCC compliant value └ Standard value for Fast mode
└ Standard value for Variant mode
NGSP compliant value └ Standard value for Fast mode
└ Standard value for Variant mode

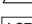
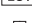
IFCC: International Federation of Clinical Chemistry and Laboratory Medicine
NGSP: National Glycohemoglobin Standardization Program

The standard values on the list may vary depending on production lots. Always check the standard values before use.

Limitations of the Procedure

Store this product at a temperature between 2°C and 8°C after dissolution, and use within 8 hours.

Glossary of Symbols

	Storage temperature limitation
	In Vitro Diagnostic Medical Device
	Manufacturer
	Biohazard
	Batch code
	Use by
	This product conforms to the Directive 98/79/EC.
	Authorized Representative in the European Community
	Consult Instructions for Use
	Catalogue number
	Date of manufacture

Materials Provided

- CALIBRATOR 90 Low (freeze dried)
Approx. 3 mL × 3 bottles
- CALIBRATOR 90 High (freeze dried)
Approx. 3 mL × 3 bottles
- CALIBRATOR 90 Diluent
10 mL × 3 bottles

Bottle materials

CALIBRATOR 90 Low: Glass with polyethylene (PE) cap and butyl rubber packing
CALIBRATOR 90 High: Glass with polyethylene (PE) cap and butyl rubber packing
CALIBRATOR 90 Diluent: Polyethylene (PE) bottle with polypropylene (PP) cap

MANUFACTURER

ARKRAY Factory, Inc.

1480 Koji, Konan-cho, Koka-shi
Shiga 520-3306, JAPAN

EUROPEAN REPRESENTATIVE

ARKRAY Europe, B.V.

Prof. J.H. Bavincklaan 2
1183 AT Amstelveen, THE NETHERLANDS

UK RESPONSIBLE PERSON

ARKRAY Ltd.

Beacon House Stokenchurch Business Park, Ibstone Road
Stokenchurch, High Wycombe HP14 3FE, UK

This product conforms to the Medical Devices Regulations 2002.

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