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【Intended use】 This product is used for QC of test process of Urine Sediment Analyzer, FUS series urinalysis hybrid and MUS series urinalysis system.

【Test principle】 Flow cytometry imaging

【Main components】 Negative control: disodium hydrogen phosphate: 6.7 mmol/L.

Note: control range varies with the batch (see the label for fixed values of each batch).

【Storage conditions and shelf life】

1. This product shall be sealed and kept in a dark place at the temperature of 2°C~8°C. For shelf life refer to the label.
2. After being unsealed for the first time, it shall be sealed and kept in a dark place at the temperature of 2°C~8°C, and the open-vial shelf life is 30 days.

【Test method】 Refer to the user manual of the applicable instrument.

【Product performance indices】

1. Accuracy: total particles number of negative control shall be $\leq 20/\mu\text{L}$.
2. Homogeneity: negative control not being tested

【Cautions】 1. This product is used for in vitro diagnostics only.

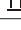
2. Shake the product for 1~3min before use, and do not shake it violently. If it is not mixed well, the number of tested particles may be smaller, thus affecting the QC result.
3. Refrigeration is prohibited.
4. To avoid or reduce relevant infection risks, please wear disposable gloves and eyes/face protection articles when using the product.

【References】

1. Ferris JA urine microscopy comparison & standard: Lab Med14: 659-662,1983.
2. Method Comparison and Bias Estimation Using Patient Samples; Approved Guideline-Second Edition.

【Instruction Approved and Modified Date】 08/2018

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 Consult instructions for use	 Temperature limit	 Manufacturer	
 In vitro diagnostic medical device	 Batch code	 Use-by date	 Biohazard
 Keep away from sunlight	 Authorised Representative	 Catalogue Number	
 Comply with In Vitro Diagnostic Devices Directive (98/79/EC)		 This Way Up	

DIRUI INDUSTRIAL CO., LTD.

95 Yunhe Street, New & High Tech.
Development Zone Changchun, Jilin 130012 P.R.China
Tel: +86 431 85100409 Fax: +86 431 85172581
E-mail: dirui@dirui.com.cn http://www.dirui.com.cn

EC REP

EMERGO EUROPE Prinsessegracht 20
2514 AP The Hague The Netherlands