MAS® UA DipTube

LIQUID ASSAYED URINALYSIS CONTROL

IVD

INTENDED USE

MAS® UA DipTube is intended for use in the clinical laboratory as a control for qualitative and semi-quantitative procedures used in routine urinalysis testing. Assay values are provided for specific systems listed. The user can compare observations with expected ranges as a means of assuring consistent performance of reagent and equipment. The product is suitable for use as a control material for physiochemical, chemical, and microscopic methods of routine urinalysis. **UA DipTube** may be used in conjunction with commercially available standardized systems for urine microscopic analysis.

PRODUCT DESCRIPTION

UA DipTube is a liquid stable control material prepared from human urine. Analyte levels are adjusted with various pure chemicals and human source materials. **UA DipTube** also contains preservatives and stabilizers.

CAUTION: **UA DipTube** is prepared from human urine pools and contains other human source material. Components of the control which are derived from human source material have been tested using FDA accepted methods and found to be non-reactive for Hepatitis B Surface Antigen (HBsAg), Hepatitis C Antibody (HCV) and HIV-1 and HIV-2 Antibody. However, no test method can offer complete assurance that products derived from human source material are free of infectious agents. This control must be handled in accordance with recommendations from the Center for Disease Control/National Institutes of Health manual, "Biosafety in Microbiological and Biomedical Laboratories," 2009.

STORAGE AND STABILITY

Refrigerated Storage (2-8°C): Store unopened **UA DipTube** at 2-8°C. Unopened tubes are stable until the expiration date on the label. Once opened, tubes of control are stable for 8 weeks or 20 reagent strip immersions, whichever occurs first.

Room Temperature Storage (18-25°C): Open tubes of control are stable 2 weeks or 10 reagent strip immersions, whichever comes first.

Do not freeze.

Bacterial contamination produces an increase in turbidity and/or a characteristic odor. Discard vial if evidence of microbial contamination is observed.

EXPECTED VALUES

Expected values may vary slightly with different lot numbers and reagent strips. Refer to the included table for values obtained on specific systems. Good laboratory practice suggests that each laboratory establish its own parameters.

Values listed were obtained using instrument(s) and instrument manufacturer's reagents and procedures at the time of assay. Subsequent modifications in instrument, reagent, or procedures may invalidate these values.

Laboratories using methods other than those listed should establish their own expected ranges.

INSTRUCTIONS FOR USE

Materials Required But NOT Provided:

- Slides
- Pipettors
- Centrifuge tubes, slides, and pipettors from standardized urinalysis systems may be used.
- Centrifuge capable of generating an RCF of 400 for KOVA method or 450 for manual methods.
- Microscope with a 10x objective and a 40x objective lens
- Urinalysis stain (Optional)
- Chemical reagent strips
- Any additional laboratory equipment or reagents required to perform the physiochemical, chemical and microscopic analyses required.

Allow the **UA DipTube** to come to room temperature. Thoroughly mix the contents of the tube before each use by gently inverting for several minutes (3 minutes minimum). Replace cap immediately and store the opened tube at 2-8°C or at room temperature (18-25°C).

Physical Tests:

1. **Appearance:** Visually inspect the color and turbidity.
2. **Specific Gravity:** Measure the specific gravity using a refractometer or reagent strip.

Chemical Tests:

1. If reading results visually, compare the reagent strip with the manufacturer's color comparison chart. If using an automated strip reader, follow manufacturer's directions for use.
2. **Reagent Strips:** Remove the cap from the tube and immerse the reagent strip into the control. Remove strip promptly and follow the manufacturer's instructions for testing. Replace cap immediately after use.
3. **Reagent Tablets or Pregnancy Cassettes:** Remove the cap from the tube and pipette the required amount of control per reagent manufacturer's instructions. Replace cap immediately after use.

Microscopic Analysis:

If using a standardized methodology for microscopic analysis, follow manufacturer's recommendations. Each laboratory should establish its own values for systems other than those listed.

If using a non-standardized methodology for microscopic analysis, centrifuge the **UA DipTube** for 5 minutes at 450 RCF (relative centrifugal force). Pour off supernatant leaving approximately 0.5 mL. Gently resuspend sediment transferring one drop onto a clean, dry microscope slide and cover with a 22 mm coverslip. Values reported are the range of microscopic elements counted in 10 hpf (high powered fields). Each laboratory should establish its own ranges.

Handle the **UA DipTube** in the same manner as patient specimens. Consistent handling of the control is necessary for uniform results.

This product contains cellular elements which may form a small amount of sediment in the bottle. To ensure consistent recoveries, invert and mix well prior to use.

The tube contains 12mL of control. **A fresh tube or a tube that has had no more than 1 reagent strip immersion should be used for microscopic analysis.** A consistent volume for testing is essential for reproducible microscopic results.

QUALITY CONTROL

All quality control requirements should be performed in conformance with local, state and/or federal regulations or accreditation requirements.

LIMITATIONS OF PROCEDURE

UA DipTube must be mixed well before each use to ensure reproducible results.

A consistent volume must be used for testing to ensure reproducible microscopic results. A fresh tube or a tube that has had no more than 1 reagent strip immersion should be used for microscopic analysis.

Organic spheres used to simulate leukocytes may not exhibit the same staining characteristics of human leukocytes.

See manufacturer's product insert for a listing of limitations for each test method. For professional use only.

TECHNICAL ASSISTANCE

In the USA, for technical assistance, call 800-232-3342 or 510-979-5417. For insert updates and information, if your laboratory subscribes to LabLink XL, visit www.maslablink.com and select LabLink Extra. Alternatively, to subscribe to LabLink XL call 800-232-3342 or 510-979-5451.

Outside of the USA, if your laboratory subscribes to LabLink XL, visit www.maslablink.com select LabLink Extra. Alternatively, please contact your local sales office or authorized distributor.

Cat. No.	Description	Size
UAT-MP	UA DipTube, Multi-Pack	10 x 12 mL

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For insert updates go to:
www.thermoscientific.com/diagnostics

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☐ LVL 1 2017-07-31
☒ LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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☐ LVL 1 2017-07-31
☒ LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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LVL 1 2017-07-31
 LVL 2 2017-07-31

Thermo
SCIENTIFIC

For Insert updates go to:
www.thermoscientific.com/diagnostics

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IVD

		LVL 1 ^(a) <R> UAT17071	LVL 2 ^(b) <R> UAT17072
KOVA (Qualitative)	Red Cells White Cells Crystals Casts	0 hpf 0 hpf Absent 0 ^(c)	1 - 62 hpf 1 - 57 hpf Absent - Present 0 ^(c)
(Quantitative)	Red Cells White Cells Crystals Casts	0 hpf 0 hpf Absent 0 ^(c)	1 - >92 per µL 1 - >92 per µL Absent - Present 0 ^(c)
Cen-Slide	Red Cells White Cells Crystals Casts	0 hpf 0 hpf Absent 0 ^(c)	1 - 65 hpf 1 - 64 hpf Absent - Present 0 ^(c)
Yellow IRIS	Red Cells White Cells Crystals Casts	** ** ** **	** ** ** **
IRIS IQ200	Red Cells White Cells Crystals Casts	0 - 5 hpf 0 - 5 hpf Absent 0 ^(c)	6 - >100 hpf 3 - 75 hpf Absent - Present 0 ^(c)
URI-system	Red Cells White Cells Crystals Casts	** ** ** **	** ** ** **
Coverslide Method	Red Cells White Cells Crystals Casts	0 hpf 0 hpf Absent 0 ^(c)	1 - 63 hpf 1 - 62 hpf Absent - Present 0 ^(c)

Acetest®, Advantus, Atlas, Clinitek® Series, Clinitest®, Ictotest®, Micro-Bumintest®, Status®, Diastix®, Reg. TM: Siemens Healthcare Diagnostic, Tarrytown, NY
 Array 360®, ICON®, Immage®, Reg. TM: Beckman Coulter Inc., Fullerton, CA
 Biovation, Reg. TM: Biovation, Inc., Hercules, CA
 Dimension®, Reg. TM: Siemens Healthcare Diagnostic, Glasgow, DE
 URL-system®, Sure-Vue, Reg. TM: Thermo Fisher Scientific, Waltham, MA
 Signify®, Contrast, OSOM Reg. TM: Genzyme Corporation, Cambridge, MA
 KOVA®, Reg. TM: Hyco Biomedical Inc., Garden Grove, CA
 DiaScreen Series, Reg. TM: Hypoguard, Minneapolis, MN
 Iris IQ200, Yellow IRIS®, Reg. TM: International Remote Imaging Systems, Inc., Chatsworth, CA
 Chemstrip®, Chemstrip® 101, Criterion, CUA, Hitachi®, Micral, Mini UA, SUA, URISYS 1100, URISYS Systems, Reg. TM: Roche Diagnostics, Indianapolis, IN
 QuickVue®, UrinQuick, Reg. TM: Quidel Corporation, San Diego, CA
 Cen-Slide®, Reg. TM: StatSpin, Norwood, MA
 PocketChem UA®, Reg. TM: ThermoFisher Scientific, Boulder, CO
 YD Uriscan®, Reg. TM: YD Diagnostics, Seoul, Korea

*** Atypical colors may be observed. Results based on intensity.**

Atypische Farben können beobachtet werden. Resultate basiert worden auf Intensität.
 On peut observer des couleurs atypiques. Résultats basés sur l'intensité.
 Si possono osservare colori atipici. Risultati basati sull'intensità.
 Pueden observarse colores anormales. Los resultados se basan en la intensidad.
 Der kan observeres atypiske farver. Resultateret baseret på intensitet.
 Er kan sprake zijn van atypische kleuren. De resultaten zijn gebaseerd op de intensiteit.
 Erapäällisillä väriä voinaavaa havaita. Tulokset perustuvat voimakkuuteen.
 Ενδέχεται να παρατηθούν απότελέσματα βασιζόνται στην ένταση.
 Det kan forekomme atypiske farger. Resultatene er basert på intensitet.
 Može wystąpić niespodziewane zabarwienie. Wyniki na podstawie intensywności.
 Podem ser observadas cores atípicas. Resultados com base na intensidade da cor.
 Наблюдаемый цвет может быть необычным. Результаты зависят от интенсивности.
 Atypiska färger kan observeras. Resultatet baserade på intensitet.
 Atipik renkler gözlemlenebilir. Sonuçlar yoğunluğu dayanır.
 可能观察到非典型颜色。结果依据强度而定。
 不規則な色が観察される場合があります。結果は明度に基づきます。

**** Data not available. If interested in participating in our value assignment process, please fax or email your contact information to our Value Assignment group at 510-771-1539, or mgc-va@thermofisher.com.**

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***** Creatinine values are based on a 1:4 dilution with water.**

Den Kreatininwerten liegt eine 1:4-Verdünnung mit Wasser.
 *** Les valeurs de créatinine sont basées sur une dilution de 1/4 avec avec de l'eau.
 *** I valori della creatinina sono basati su di una soluzione di 1 a 4 con acqua.
 *** Los valores de la Creatinina se basan en una dilución de 1:4 con agua.
 *** Kreatininværdiene er baseret på en 1:4 fortyndning med vand.
 *** De creatininwaarden zijn gebaseerd op een 1:4-verdunning met water.
 *** Kreatiiniarvot perustuvat vedellä laimentamiseen 1:4.
 *** Οι τιμές κρεατινίνης βασίζονται σε μια αράση 1:4 με νερό.
 *** Kreatinin-verdier er basert på en 1:4-fortynning med vann.
 *** Wartości kreatyniny dotyczą roztworu wodnego sporzązonego w stosunku 1:4.
 *** Os valores de creatinina são baseados numa diluição com água de 1:4.
 *** Значения для креатинина приведены для случая разбавления водой в пропорции 1:4.
 *** Kreatininvärdena baseras på spädningen 1:4 med vatten.
 *** Kreatinin değerleri suyla 1:4 oranında seyreltmeye dayanır.
 *** 依据 1:4 水稀释液获得肌酐值。
 *** クレアチニン値は水による 1:4 希釈に基づきます。

****** Sulfosalicylic Acid (3%) values are based on a 1:3 dilution with water.**

Werte der Sulfosalicylic Säure (3%) basieren auf einer 1:3 Verdünnung mit Waser.
 **** Des teneurs en acide de Sulfosalicylique (3%) sont basés sur une dilution de 1:3 avec de l'eau.
 **** Gli indici dell'acido di Sulfosalicylico (3%) sono basati su una diluizione di 1:3 con acqua.
 **** Los valores del ácido de Sulfosalicilico (3%) se basan en una dilución de 1:3 con agua.
 **** Værdierne for Sulfosalicylyre (3 %) er baseret på en 1:3 fortyndning med vand.
 **** De sulfosalicyluurwaarden (3%) zijn gebaseerd op een 1:3-verdunning met water.
 **** Sulfosalisylylhapon (3 %) arvot perustuvat vedellä laimentamiseen 1:3.
 **** Οι τιμές σουλφοσαλικυλικού οξέος (3%) βασίζονται σε αράση 1:3 με νερό.
 **** Verdier for sulfosalicylyre (3 %) er basert på en 1:3-fortynning med vann.
 **** Wartości kwasu sulfosalicylowego (3%) dotyczą roztworu wodnego sporzązonego w stosunku 1:3.
 **** Os valores de ácido sulfosalicílico (3%) são baseados numa diluição com água de 1:3.
 **** Значения для сульфосалициловой кислоты (3 %) приведены для случая разбавления водой в пропорции 1:3.
 **** Värdena för Sulfosalidlysyra (3 %) baseras på en spädning 1:3 med vatten.
 **** Sulfosalisilik Asit (3%) değerleri suyla 1:3 oranında seyreltmeye dayanır.
 **** 依据 1:3 水稀释液获得磺基水杨酸 (3%) 值。
 **** スルホサリチル酸 (3%) の値は、水による 1:3 希釈に基づきます。

******* No claim is made for expected value nor the stability of these constituents.**

Zu diesen Bestandteilen werden keine Angaben über Zielwerte oder Stabilität gemacht.
 ***** Aucune information n'est indiquée concernant la valeur attendue ou la stabilité des constituants.
 ***** Nessun reclamo si fa ne' per i valori attesi, ne' per la stabilità' dei componenti.
 ***** No se han establecido los valores esperados o la estabilidad de estos constituyentes.
 ***** Der er ikke angivet nogen forventet værdi eller stabilitet for disse bestanddele.
 ***** Wij bieden geen enkele garantie ten aanzien van de verwachte waarde of de stabiliteit van deze bestanddelen.
 ***** Mitään välttää e sitetään näidenaineosien odotetuista arvoista tai stabiliudesta.
 ***** Δεν πραγματοποιείται καμία αίσχηση για την αναμενόμενη τιμή, ούτε για τη σταθερότητα των συστατικών αυτών.
 ***** Det gis ikke ingen garantier for forventet verdi eller for disse konstituentenes stabilitet.
 ***** Nie deklaruje się oczekiwanej wartości ani stabilności następujących składników.
 ***** Não é feita qualquer declaração quanto ao valor esperado ou à estabilidade destes componentes.
 ***** Не делается никаких заявлений относительно ожидаемого количества или стабильности этих компонентов.
 ***** Det görs dock inga anspråk angående förväntade värden eller stabilitet för dessa beständsdelar.
 ***** Ancak, nebeklenen değer için ne de bu bileşenlerin stabilitesi için tahminde bulunulmamıştır.
 ***** 这些成分的预期值和稳定性都没有被提及。
 ***** これらの成分の予測される測定値または安定性に対しては保証しません。

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If you have any questions concerning this notification, in the USA please contact the Technical Support Department at 800-232-3342 or 510-979-5417. Outside the USA, please contact your local Subsidiary or Distributor.

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Microgenics Corporation
Microgenics GmbH